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DILANAR® INSTRUCTIONS FOR USE

Medical Device to narrow the nasal columella and expand the nasal valve.

The patented and unique design of the product, narrowing the columella and widening the nose wings, allows better nasal breathing; It is also built with soft and comfortable material.

INDICATIONS

Helps improve nasal breathing:

- during normal breathing
- while sleeping (helps prevent snoring)
- while practising sports (reduces nasal resistance)

INSTRUCTIONS FOR APPLICATION

Insert Dilanar into the nasal vestibule as shown in the figures on the box supplied.

When using Dilanar for the first time, stand before a mirror so as to make it easier for you to insert the product into the nasal vestibule.

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Replaced: If properly stored and its package is not damaged, Dilanar can be used until it is no longer efficient.

Whith this: With regular use, if properly cleaned and stored in a dry space protected from light and heat, the device can be used for approx. 3 months. When the elasticity or ability to adapt to the nasal vestibule of Dilanar decreases it loses its effectiveness for which it should be replaced.

PRECAUTIONS

- For personal use only. Use restricted to the nasal vestibule only.
- Do not use if there are wounds or irritations in the nasal cavities.
- Do not exceed 24 running hours of use.
- Stop using the device, if you experience discomfort.
- It is also used in children under medical advice.
- Keep out of the reach of Young children only.

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